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**RELEASE SINUS
IN 15 MINUTES**

Information leaflet

INDICATION: For the treatment of rhinitis, rhinosinusitis and nasal congestion.

**Decongests the nose nearly instant
15 ml nasal spray (approx. 125 sprays)**

MODE OF ACTION: When sprayed on the nasal mucosa, Sinulan Express Forte forms a resistant, osmotic yet non-irritant protective film over the nasal surface. This film attracts hypotonic liquid from the deeper parts of the mucosa and creates a stream of hypotonic fluid from the inside outwards. This mechanical mode of action detaches and drains the contaminants present in the nasal cavity including free-floating protein molecules, virus particles and bacteria, and decongests the nasal passage. The osmotic pressure also helps to open & clean the sinuses, thereby reducing intra-sinus pressure and pain. The film formed by Sinulan Express Forte protects the nasal surface from dryness, irritation and external aggressions. This exclusively topical, mechanical and instant mode of action acts locally without being absorbed in the body and helps provide quick relief. Clinical results show noticeable reduction in nasal congestion and general sinusitis symptoms, thereby reducing the need for antibiotics.

DIRECTIONS:

- 1 - Blow your nose thoroughly before using the product
- 2 - Shake the bottle
- 3 - Remove the protective cap
- 4 - Press the sprayer once or twice until a fine mist of product appears
- 5 - Insert the nozzle into one nostril and press on the sprayer to apply 2-3 sprays in each nostril
- 6 - Clean the nasal insert with tissue paper and close the bottle with the cap after each use

Children will need the help of an adult.

Apply 3-4 times per day maximum, until complete recovery.

PRECAUTIONS: Sinulan Express Forte is for topical use in the nasal cavity only. Read the instructions leaflet carefully before use. Respect the directions. Wash hands before use. To reduce risk of cross infection, each tube must be used by one patient only. The mode of action of Sinulan Express Forte is due to instant osmosis and may induce slight tingling within the first 10-15 minutes following each product application. However, if the irritation is too strong or persists for a long period of time, we recommend that you stop using the product as you may be allergic to one of the ingredients used to prepare Sinulan Express Forte. Sinulan Express Forte acts within 10-15 minutes after application with a significant nasal discharge. Do not reapply Sinulan Express Forte or use any other nasal treatment during this time. The nasal discharge can last up to 4-6 hours depending on the severity of the initial nasal congestion. Seek medical advice if the symptoms persist with no improvement after 7 days of use of Sinulan Express Forte.

STORAGE: Keep out of reach and sight of children. Store at room temperature. Do not refrigerate or freeze. Do not use after the expiry date, more than 30 days after 1st use, or if the container is damaged.

COMPOSITION: Aqua, Glycerol (26.14%), Hydroxypropylcellulose, and association of essential oils of *Eucalyptus globulus*, *Mentha piperita*, *Rosmarinus officinalis* and *Thymus satureioides*.

INTERACTIONS: Due to Sinulan Express Forte composition and to its mechanical & topical mode of action, the probability of interaction with any systemic treatment is unlikely. Do not apply any other topical nasal treatment 15 minutes before or after the use of Sinulan Express Forte.

CONTRAINDICATIONS: Check the Sinulan Express Forte composition and do not use if you are sensitive or allergic to any of the ingredients. Although all the ingredients used to prepare this product are derived from food ingredients and used in very small quantities, in the absence of sufficient specific safety data, we do not recommend the use of this product in children below the age of 12 and in patients suffering from asthma, rhinosinusitis of fungal or allergic origin and in cases of severe respiratory conditions. We recommend asking for medical advice before using this product during pregnancy and breastfeeding.

SIDE EFFECTS: Sinulan Express Forte is not dangerous if swallowed. Avoid contact with the eyes. In case of contact, flush thoroughly with water. If you observe any type of allergic reaction in the minutes following Sinulan Express Forte application, cease using Sinulan Express Forte and flush your nose thoroughly with clear fresh water, and if necessary, consult a medical professional. If you notice any serious incidents due to the use of Sinulan Express Forte, please notify immediately the manufacturer/distributor and national competent authorities.

The packaging (including the container) can be disposed of as household waste after use.

Leaflet version/revision: NHE b4-0120

MANUFACTURER

 VITROBIO SAS, ZAC de Lavour, 63500 Issoire, France

MADE in France

Marketing authorization holder/Importer:

STADA (Thailand) Co., Ltd, 8 T One Building, 18th Floor, Unit 5, Soi Sukhumvit 40, Sukhumvit Road, Phra-khanong, Klongtoei, Bangkok 10110 Thailand

E-mail: stadaasiapv@stada.com.hk

Tel: +66 2059 7420